

# KN95 Quick Guide

KN95 masks, or other foreign respirators, are not certified by U.S. government standards for respiratory protection.

KN95 masks are to be used as a form of face covering, such as a cloth or disposable mask, for source control.

Do not use as a substitute for respiratory protection or surgical masks.

Discard after single use



# How to put on a KN95 face mask



Inspect the mask prior to use, discard if damaged or soiled. Pull the elastic ear loops behind your ears and adjust the mask to fit over your nose and mouth.



Press with both index fingers from the middle to the sides, making its upper end close to the nose bridge.



Check for fit. The mask should fit snugly and comfortably.



Cover the mask with your hand and exhale. Adjust the mask if you feel air leaks. Keep adjusting



Questions? Contact the Department of Environmental Health and Safety.

Mike Courtner Training Manager

212-998-1341

EHS@NYU.EDU

Make sure to thoroughly wash your hands before putting on or taking off any face covering!